



IMS® WRAP

SPORECHECK™

BAGETTE®
POUCHES

STERILIZATION SOLUTIONS

| Products and Guidelines for Enhanced Office Safety

How the best perform




STERILIZATION MONITORING


CDC recommends placing an indicator inside each pack to be sterilized and to conduct biological monitoring of your sterilizer at least weekly.

STERILIZATION STEAM INDICATORS

- Indicator strip verifies that steam sterilization parameters of time and temperature have been met.

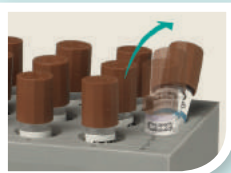

CONVENIENT, BUILT-IN
VIAL CRUSHER

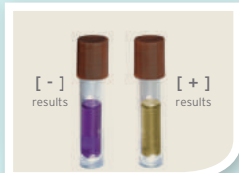

TEST RESULTS ARE EASY TO
INTERPRET WITH A VISIBLE COLOR
CHANGE FOR POSITIVE RESULTS,
LEAVING NO ROOM FOR ERROR

STEAM STERILIZATION INTEGRATOR

- Type 5, multi-variable sterilization monitor designed to react to all critical variables in the sterilization cycle: time, temperature and presence of steam.
- Utilizes a moving front indicator technology, which is an advancing color bar moving from reject to accept, eliminating the need to interpret or match colors.

Unprocessed



Processed - REJECT



Processed - ACCEPT



SPORECHECK 24-HOUR IN-OFFICE BIOLOGICAL MONITOR SYSTEM

- In-office biological monitoring allows you to take control of your steam sterilizer testing and provides you with the peace of mind that your sterilizer is functioning properly with results in 24 hours.
- The SporeCheck system is easy to use, and the biological indicator (BI) vials work for ALL types of steam sterilizers, including gravity, pre-vacuum and immediate use sterilization.
- Helps you follow CDC guidelines, which recommend that steam sterilizers be monitored at least weekly, preferably daily, as well as with every load that contains implantable devices.

STERILIZATION PACKAGING


CDC recommends all items to be sterilized should be packaged using an FDA cleared container system or wrap that is compatible with the sterilization process used.

AUTOCLAVE AND UNIVERSAL CASSETTE WRAP

- Engineered to withstand the harsh conditions of sterilization.
- Unique compilation of textiles is designed to be strong and soft for easy wrapping and durability.
- Low-lint, memory free wrap is available in two grades to meet your needs: Autoclave and Universal.

BONDED DOUBLE WRAP

- Two sheets of wrap ultrasonically bonded together on two edges for quick and easy wrapping application, assuring compliance.
- Two different colors sheets to easily identify breaches post sterilization.
- High strength and durability aid in resistance to tears and punctures, protecting contents of the package during storage and transportation.
- Made of polypropylene – any unused or unsaturated wrap can be recycled as a recycle code of "5".
- Compatible with many common sterilization cycles.
- Product part code and size printed on each sheet for quick and easy identification when reordering product.


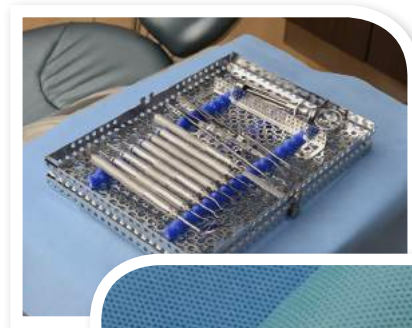


WRAP SIZE RECOMMENDATIONS

	12" x 12"	15" x 15"	20" x 20"	24" x 24"
Signature Series® Cassettes				
Exam, Double-Decker® 8-Instrument, Ultrasonic Insert, Symmetry IQ® Tip & Handpiece Cassette	X	X		
Space Saver Seven™ Cassette	X	X		
Double-Decker 14-Instrument Cassette Δ		X	X	
Small 8 or 10 Instrument Cassette Δ		X	X	
Signa-Stat or Double-Decker 20-Instrument Cassette			X	
Large 16 or 20 Instrument Cassette			X	
Oral Surgery Cassette				X
Orthodontic Cassette Box				X
Large Thin Orthodontic Cassette			X	
Small Thin Orthodontic Cassette			X	
Double-Decker Large Ortho Cassette			X	

Δ The Double-Decker 14-instrument cassette and Small cassettes are best with the 15" x 15" sterilization wrap. However, if an office is trying to purchase only one size, the 20" x 20" size is recommended.

WRAPPING TECHNIQUES

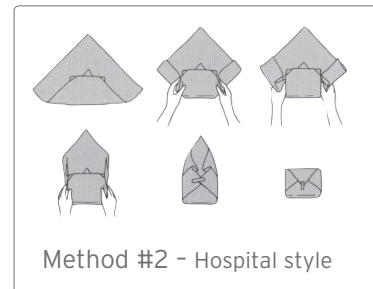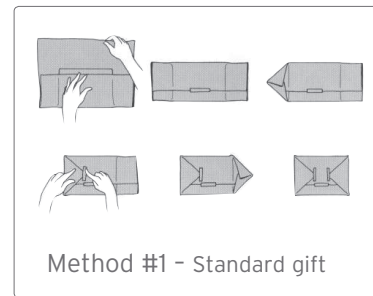

EXAM



STANDARD IMS MONITOR TAPE
PRE-PRINTED WITH SPECIFIC PROCEDURAL
SET-UPS, COLOR-CODED OR BLANK.



IMS LEAD-FREE MONITOR TAPE
SAFE FOR THE ENVIRONMENT, YOUR STAFF
AND YOUR PATIENTS AND DOES NOT REQUIRE
SPECIAL HAZARDOUS DISPOSAL.



IMS TAPE DISPENSERS
RUBBER SUCTION CUPS ON THE BASE
SECURE TO TABLE OR COUNTER. CAN
ALSO BE MOUNTED UNDER CABINET.

IMS MONITOR TAPES AND DISPENSERS

- Strong 3/4" (19 mm) tape can be written on and visually differentiates processed from non-processed cassettes
- Pre-printed and color coded tapes help organize and easily identify wrapped packs
- Easy to use and read results
- 60 yards (55 m) per roll
- For steam autoclaves



BAGETTE® STERILIZATION POUCHES

Internal and external indicators for both Steam and EO Sterilization. Responds to all variables of sterilization: time, temperature & presence of steam

Thumb notch for ease of opening

High quality medical grade paper and multi-layer blue tinted film provide strength and durability

Thick, strong thermal seal to protect against cassette and instrument breaches

Wide self-seal adhesive strip to provide reliable seal

Pouches are lead free and are not made with natural rubber latex

BAGETTE USAGE CHART


PART CODE	SIZE	SUITABLE CONTENTS
IMS-1346	2-1/4" x 4" (57 mm x 102 mm)	Rubber dam clamps or burs
IMS-1347	2-3/4" x 9" (70 mm x 229 mm)	1 handpiece or 1-3 hand instruments
IMS-1236	3-1/2" x 9" (89 mm x 229 mm)	2-5 hand instruments
IMS-1348	5-1/4" x 10" (133 mm x 254 mm)	Handpiece or multiple instruments
IMS-1237	7-1/2" x 13" (190 mm x 330 mm)	• Exam Cassette*
IMS-1238	10" x 14" (254 mm x 356 mm)	• Small Cassette*
IMS-1239	12" x 17" (305 mm x 432 mm)	• Large Cassette*
IMS-1345	13" x 20" (330 mm x 508 mm)	• Oral Surgery Cassette*

*Please see Hu-Friedy.com for complete list of suitable cassettes for this pouch.


This color change guide is used to aid in the interpretation of the color change that occurs when the pouches are processed.

Ideally, the indicator should change color from the first box to the last box, but due to the large variation in sterilization chambers, this is not always the case. Therefore, below is a suggested acceptable range of color change.

STEAM STERILIZATION

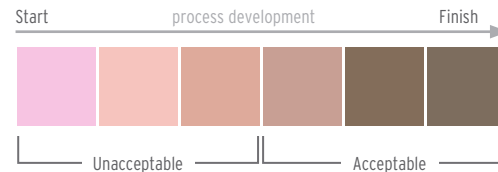

EO STERILIZATION

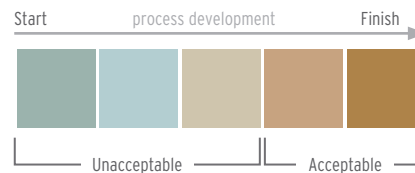

STERILIZATION PRODUCTS

SporeCheck 24 hour in-office steam sterilizer starter kit IMS-1376

Includes: Record Notebook, 1 Dry Block Incubator with built-in vial crusher and 1 SporeCheck BI (Box of 25)

SporeCheck In-Office Test (25 Tests/100 Tests) IMS-1373 IMS-1373H

SporeCheck 55-60° Dry Block Incubator IMS-1374

SporeCheck Sterilizer Record Notebook IMS-1375

Bagette Pouches

2-1/4" x 4" 200 pouches IMS-1346

2-3/4" x 9" 200 pouches IMS-1347

3-1/2" x 9" 200 pouches IMS-1236

5-1/4" x 10" 200 pouches IMS-1348

7-1/2" x 13" 100 pouches IMS-1237

10" x 14" 100 pouches IMS-1238

12" x 17" 100 pouches IMS-1239

13" x 20" 100 pouches IMS-1345

Autoclave Wrap

24" x 24" 500 sheets IMS-1210

20" x 20" 500 sheets IMS-1212

15" x 15" 1,000 sheets IMS-1211

12" x 12" 1,000 sheets IMS-1213

Universal Wrap

24" x 24" 500 sheets IMS-1214

20" x 20" 500 sheets IMS-1216

15" x 15" 1,000 sheets IMS-1215

12" x 12" 1,000 sheets IMS-1217

Bonded Double Wrap

24" x 24" 240 pieces IMS-2224

20" x 20" 480 pieces IMS-2220

15" x 15" 480 pieces IMS-2215

Chemical Indicators

IMS Steam Sterilization Indicators (250 per box) IMS-1240

IMS Steam Sterilization Integrators (100 per pouch) IMS-1241

IMS Monitor Tape: 3/4" (19mm)

Ortho IMS-1243 Cement IMS-1253 Orange IMS-1263

Archwire IMS-1244 Composite IMS-1254 Purple IMS-1264

Banding IMS-1245 Endo IMS-1255 Yellow IMS-1265

Bonding IMS-1246 Exam IMS-1256 Red IMS-1266

Restorative IMS-1247 Extraction IMS-1257 Green IMS-1267

Surgery IMS-1248 Perio IMS-1258 Blue IMS-1268

X-Ray IMS-1249 Prophy IMS-1259 Black IMS-1269

Amalgam IMS-1251 Pedo IMS-1260 Lead Free Blank IMS-1460


C&B IMS-1252 Gray IMS-1262 Blank IMS-1250

Tape Dispenser (Holds up to 9 rolls) IMS-1270

Mini Tape Dispenser (Holds up to 4 rolls) IMS-1270H

Implementing an infection prevention and instrument processing protocol that follows the CDC Guidelines can help your practice provide a safer and more comfortable environment for your staff and patients. Hu-Friedy Infection Prevention Products and Instrument Management System help you do just that.

Your Hu-Friedy Representative is always available to help and answer any questions you may have. Call 1-800-Hu-Friedy or visit Hu-Friedy.com for more information.

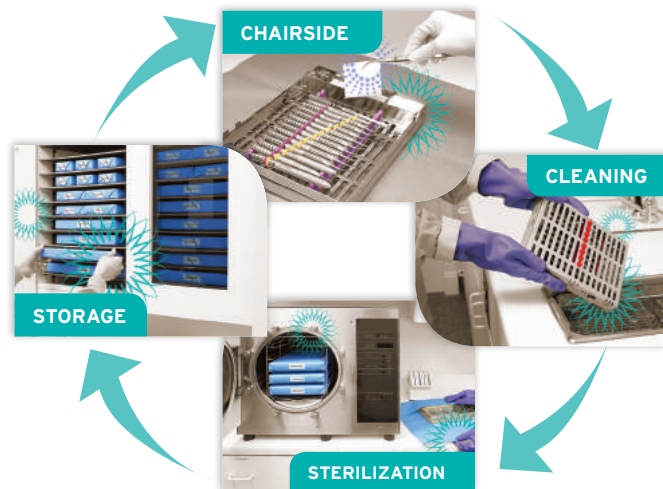

For more information about Hu-Friedy's Total Solutions visit **HU-FRIEDY.COM**

PREVENTIVE



RESTORATIVE



ORTHODONTIC



INSTRUMENT MANAGEMENT



ENDODONTIC



SURGICAL



POWER



Hu-Friedy Mfg. Co., LLC: 3232 N. Rockwell St. | Chicago, IL 60618 | USA
Telephone: 1-800-Hu-Friedy (1-773-975-6100) | Website: Hu-Friedy.com

©2013 Hu-Friedy Mfg. Co., LLC. All rights reserved. [200] IMS-306/1018

Hu-Friedy