# Bur Cushion

IMS-1372S/T • IMS-1372SH/TH

For intended use only by dental professionals. Inspect, clean and sterilize before each use. Steam sterilize for at least 4 minutes at 270°F/132°C or 30 minutes at 250°F/121°C. Recommend 30 minute dry time after sterilization cycle. Do not heat above 350°F/177°C. Use recommended chemicals only. Do not disinfect with phenols. Detailed care and maintenance instructions available upon request or go to Hu-Friedy.com.

#### Directions for use:

- Remove stainless steel lid from silicone base by sliding the lid away from the base.
- Place the bur shanks into the base at the “x’s”.
- Insert burs deep enough so the tips do not touch the lid.
- Slide lid back onto the base along the grooves.
- Place the Bur Cushion inside the IMS cassette and process instruments.  
(The Bur Cushion can be ultrasonically cleaned and sterilized without use of IMS cassettes. Follow the cleaning and care instructions from the bur manufacturer.)

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#### Notes:

- It is normal if burs are at slight angles when held within the Bur Cushion.
- Burs are gripped in the base. The lid is provided to protect the burs and other instruments within the IMS cassette.
- Burs are cutting instruments and should always be handled with care. Touch only the shank of the bur. Use a hemostat or dressing pliers to assist in handling burs.
- If the silicone base of the Bur Cushion discolors, wipe with isopropyl alcohol, then rinse thoroughly.
- If burs are carbon steel, a milk bath is recommended if using steam sterilization.

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Hu-Friedy Mfg. Co., LLC