Pantone 326C  
Pantone 1945C  
Pantone 425C  
Black

18.5 inches Width

# ENZYMAX EARTH<sup>®</sup>

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For professional use only. Shake well before using. Use with water 90°-140°F (32°-60°C), demineralized water is ideal.

**Presoak:** Add 1 fl. oz. (30ml) of concentrate per 1 gallon (3.8L) of water. Soak for 3-5 minutes. Rinse thoroughly. **Linen Presoak:** Add 2 fl. oz. (60ml) per 1 gallon (3.8L) of water. Soak clothing for 10 minutes. Proceed with normal washing. **Ultrasonic Cleaning:** Add 1 fl. oz. (30ml) of concentrate per 1 gallon (3.8L) of water. After cleaning, rinse and dry thoroughly. **Evacuators:** Add 2 fl. oz. (60ml) per 1 gallon (3.8L) of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly.

**Store in a cool, dry place. Change daily or when visibly soiled.**

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**International :** 1-352-323-3500

**DANGER**

Causes serious eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled.

Avoid breathing dust/fume/gas/mist/vapours/spray. Wear eye protection/face protection. Wear respiratory protection. **IF INHALED:** If breathing is difficult, remove person to fresh air and keep comfortable for breathing. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. If experiencing respiratory symptoms: Call a POISON CENTER/doctor. Dispose of contents/container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation.

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**Manufactured for | Fabriqué pour :** Hu-Friedy Mfg. Co., LLC